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Referanslar

- Goldie, J. Et al, 1989. Media fort he culture of Campylobacter pylori. Am. J. Gastroenterology, 84:79
- CDC, Helicobacter pylori, july 1998
- Histology compared with chemical testing for urease for rapid detection of Helicobacter pylori in gastric biopsy specimens; Clin Pathol 1993;46:684685

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ÜRETİM YERİ

Intended Use

HelicoMax^{plus} is used to qualitatively monitor the urease enzyme activity of *Helicobacter pylori* in gastric mucosal biopsy samples. This is an in vitro diagnostic medical device for professional use only.


General Information

Helicobacter pylori are spiral shaped gram-negative bacteria infecting mucosa of the stomach. Epidemiological, experimental and clinical studies showed that the bacteria are related with acute and chronic gastritis, intestinal metaplasia, peptic ulcer and stomach cancer pathogenesis.

Helicobacter pylori survives in the stomach by neutralizing stomach acid by means of hydrolyzing urea with its urease enzyme. The urease enzyme of the microorganism has a unique structure, it has higher affinity to its substrate and has higher activity compared to other ureases.

Principle of the Test

This product was developed in order to detect the urease activity of *H. pylori* in stomach biopsy specimens. Urease in the biopsy specimen catalyzes hydrolysis of urea present in the test device. This reaction produces ammonia which leads to an increase in pH of the medium. Increased pH turns the color of the indicator ring from yellow to reddish/pinkish color. The formula of the reaction is as follows: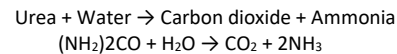

Box Content

01-001 HelicoMax^{plus} Test Devices (50 pieces)
Instruction For Use

Storage and Stability

Kit can be stored refrigerated at room temperature (2-25°C) until its expiration dated printed on the label. Shelf life is 24 months after the production. Do not use products with damaged packaging. Do not use if outer ring is not yellow

Warnings and Precautions

- Read the instructions before using the product.
- This is an in-vitro diagnostic device to be used only by competent and professional staff.
- Do not use past its expiration date.
- Do not use if packaging is damaged.
- Do not use if ring color is not yellow.
- *H. pylori* colonization in the stomach can be patchy, so the original location of the biopsy specimen is important. Sometimes it may be necessary to use two or three biopsies.
- *H. pylori* colonization can be altered if the patient takes proton pump inhibitors. This case can decrease accuracy of antral biopsy.
- Bloody biopsy specimens may lead to faulty results.
- You should practice with maximum attention as patient samples can be infectious.
- As with all diagnostic tests, the final diagnosis should not be made depending on only one test result. The final diagnosis should be made by a physician based on other clinical and laboratory findings.

Collection of Samples

After the endoscopic examination of the stomach the biopsy specimen should be taken as usual. Biopsy specimen should be taken from pyloric antrum or greater curvature. In some cases *H. pylori* colonization can be patchy and they can be present at other parts of the stomach. Two or three biopsy specimens can be taken from different parts if suspicion arise.

Test Procedure

1. Write down the patent information and date on the label. Peel the label halfway to uncover the specimen well. Be careful not to displace the content.
2. Place the biopsy specimen(s) on the yellow circle. Recover the well with the label carefully. The specimen should be between the yellow pad and the plastic button.
3. Press with your finger against the plastic button under label so the specimen underneath is squeezed. Place the test device on a level surface and start the timer.
4. Observe the color change of the yellow outer ring. Use the inner circle for yellow reference to color change. Depending on the concentration of *H. pylori* in the specimen the color change from yellow to reddish may begin within seconds.


Interpretation of the Results

Positive Results:

The result is positive if the outer ring's color changes from yellow to reddish/pinkish. The severity of the infection can be guessed depending of the speed of the reaction and the intensity of red color. The inner circle remains yellow and serves as a reference to color change. Positive result in less than 1 minute.

Negative Result:

If the outer circle remains yellow and there is no change to red then the result is negative. Wait for 60 minutes to report negative result.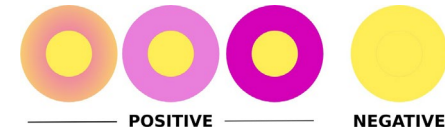

Performance of the Test

*** The sensitivity of the test is 99% and the specificity of is 97%.

References

1. Goldie, J. Et al, 1989. Media fort he culture of *Campylobacter pylori*. Am. J. Gastroenterology, 84:79
2. CDC, *Helicobacter pylori*, july 1998
3. Histology compared with chemical testing for urease for rapid detection of *Helicobacter pylori* in gastric biopsy specimens; Clin Pathol 1993;46:684685

Manufacturer and Symbols

	In Vitro Diagnostic Device
	CE Mark
	Read the instructions before use.
	Store between defined temperature range.
	Do not reuse the test.
	Reference/Catalog number of the product
	LOT/Batch number of the product
	Test quantity in the box
	Do not use if packaging is damaged.
	Expiration date
	Manufacturer

Innomedbio
Innomed Biotechnology
Hamidiye Mah. Gürbüz Sk. No: 19/A Çekmeköy, İstanbul, Türkiye
info@innomedbio.com/www.innomedbio.com

MADE IN
Türkiye
HelicoMax^{plus}