

In vitro diagnostic medical device according to Regulation (EU) 2017/746 (IVDR)

H. PYLORI UREASE TEST CASSETTE

For Rapid Detection of Helicobacter pylori in Gastric Biopsies

HelicoMax^{plus}: The Newest

Helicobacter Pylori Test

While there are several methods of detecting H. pylori, biopsy testing using rapid urease is considered the "Gold Standard" by professionals worldwide.

HelicoMax Plus Rapid Urease Test (RUT) is the most effective and easy to use Rapid Urease Test on the market today. HelicoMax Plus RUT offers significant improvements that make detecting H. pylori infections in gastric mucosal biopsies faster, easier, less expensive and more accurate than ever before.




FEATURE	ADVANTAG
No Refrigeration Required	Can be stored at room temperature. No need to keep temperature logs.
Rapid Results	Positive results in as little as 1minute, negative results in 1 hour
Accurate	99% sensitivity, 97% specificity
Dry Test	No incubation or reagents required
Cost Effective	Low per test cost
2 Year Shelf Life	Never worry about product dating

1 HOUR
RAPID
urease test
for the
detection of
H. pylori

6 Simple Steps

1. Peel off the label slowly

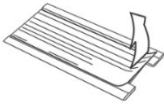


2. Place one or more biopsies on test



3. Seal the label tightly.



4. Press the push button.

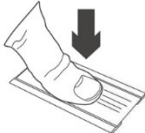

5. Wait up to one hour



6. Read results

Red = positive

Yellow = negative



Product No.	Description	Packaging
01-001	HelicoMax Plus Rapid Urease Test	50 tests / box (control included)
01-003	HelicoMax Plus Urease/Glycerin Controls Solution	3 mL dropper bottle